REDAX®

IT

SERBATOIO IN SILICONE CON VALVOLA ANTI-REFLUSSO

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DESCRIZIONE

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EN

SILICONE RESERVOIR WITH ANTI-REFLUX VALVE

INSTRUCTIONS FOR USE

DESCRIPTION

The drainage system is composed of a soft silicone reservoir fitted with an anti-reflux valve and a connector for attachment to the drainage tube. The reservoir may have a capacity of 100, 200 and 400 cc. The system is set up by connecting the drainage tube, selected according to clinical requirements, to the reservoir connector creating the vacuum by simple compression. Thanks to the typical elasticity of silicone, the drainage suction intensity remains constant until the reservoir is completely full.

Note: This device can only be used in safety conditions in the field of applications and using the procedures indicated on this operating instruction sheet for the specific type of product. The

DRENTECH® PUMP

manufacturer declines all liability relating to improper use or use different to that indicated.

PRECAUTIONS

There are no known contraindications. Do not use if the package has been opened or damaged. Avoid exposure to high temperatures and ultraviolet rays during storage. For discarding and disposal of the device, adopt suitable precautions and act in accordance with the provisions of the law in force for biologically hazardous waste. Ethylene Oxide gas sterilized. The product is sterile if the packaging is undamaged. Single-use device. Reuse may lead to alteration of performance and risks of cross-contamination. Discard after each single use, do not re-use.

INSTRUCTIONS FOR USE

Figure 1: Connect the free end of the drainage tube (A) to the conical connector (B) of the silicone reservoir.

Figure 2: Holding the silicone reservoir in the palm of the hand, start a moderate pressure until full compression.

Figure 3: When reservoir compression has been completed, close the drain valve (C) with the special cap. When the hand that was compressing the reservoir is released, suction to the drainage tube will immediately be activated.

Figure 4: Caution: Do not compress the silicone reservoir by pressing from the bottom. This would cause deactivation of the suction system.

LATEX-FREE MEDICAL DEVICE.

Date of issue of the last version :
see last page : (REV.: XX-XXXX)

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