**REDAX**®

**IT**

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#### DESCRIZIONE

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### INSTRUCTIONS FOR USE

#### DESCRIPTION

The surgical drainage is a sterile and single-use device. It is available in numerous versions which differ between them in the profile of the tube segment and/or in the material of which they are made (silicone or PVC). All the versions are available in different sizes and are supplied in double sterile packaging. Versions are available with the trocar needle already mounted to the drainage in order to facilitate positioning with aseptic technique. All the drainages have depth markings, graduated from the end of the perforated section: these markings facilitate accurate positioning of the drainage in the wound. All the drainages, both silicone and PVC, are biocompatible and hemocompatible (in accordance with the regulations in force). All the subject products are compatible for use with gravity or suction drainage systems.

#### INDICATIONS

The surgical drainages are used in numerous different fields of application with the purpose of evacuating potentially pathological fluids and blood that has accumulated in natural and/or newly-formed cavities following surgery. All the surgical drainages are compatible for use in conjunction with postoperative blood recovery systems with the purpose of autotransfusion.

**Note:** This device can only be used in safety conditions in the field of applications and using the procedures indicated on this operating instruction sheet for the specific type of product. The manufacturer declines all liability relating to improper use or use different to that indicated.

#### SUGGESTIONS FOR USE

Positioning the drainage.

- Before positioning the drainage, carefully select the type, the position and the size required. These parameters depend on the type of operation, the physiology of the patient and the experience of the physician.
- Using an aseptic technique, carefully remove the sterile thoracic drainage from its protective wrapping.
- Position the drainage in the wound using an aseptic procedure. With products fitted with trocar needle, use the latter according to the usual positioning technique, working from the inside to the outside.
- For proper positioning use the depth markings indicated on all the drainages.
- Fix the drainage by means of a stitch or plaster.
- Connect the positioned tube to the selected drainage system following the instructions on the relevant instruction leaflet. In the case of drainages fitted with trocar needle, cut the latter and dispose of it before connecting to the drainage system. If necessary, the connector provided in the package can be used (where provided) to make the connection to the drainage system.

Removing the drainage.

- Remove the tape or stitch or any other type of fixing.
- Disconnect the drainage system.
- Remove the drainage by pulling slightly and immediately apply a wound dressing.

#### CONTRAINDICATIONS

There are no known contraindications.

#### GENERAL PRECAUTIONS

- The product is sterile unless the package has been opened or damaged. Do not use if the package has been opened or damaged.
- Exposure to elevated temperatures and ultraviolet light should be avoided during storage.
- For discarding and disposal of the device, adopt suitable precautions and act in accordance with the provisions of the law in force for biologically hazardous waste.
- Single-use product. Reuse may lead to alteration of performance and risks of cross-contamination. Dispose of the device immediately after use. Do not reuse.

#### STERILE - Ethylene oxide gas sterilized.

#### MATERIALS USED

PVC and silicone biocompatible and hemocompatible.

LATEX-FREE MEDICAL DEVICE.

#### WARNINGS / PRECAUTIONS FOR USE

- Do not manipulate the drainage in any way nor make additional holes with the aid of cutting implements.
- If fixing with a stitch, carefully avoid stitching the tube itself or to cut it with needles or sharp edges. Avoid a too tight stitch which might cause the drainage to rip.
- After positioning the drainage, ensure that the perforated section is completely inside the wound and that the connections are perfectly sealed.
- The drainage must be removed by hand, pulling slightly and avoiding sudden movements. Avoid the use of metal instruments such as forceps or devices which might cause rupture of the drainage.
- If the drainage is left in the site for very long periods of time, it might be difficult to remove it. Greater attention must therefore be paid during removal.
- After removal ensure complete integrity of the drainage, an essential condition to exclude that drainage fragments caused by accidental rupture remain in the cavity.
- It is recommended not to use the drainage tube for a period of more than 29 days.
- Do not use roller clamps or similar metallic milking device on silicone tubes.
- Do not use any abrasive tools, such as cotton flock or gauzes. Milking the silicone tubes by means of those tools can make damages on drain surface reducing the mechanical resistance.
- Do not use any alcoholic or aggressive liquid to lubricate the surface of the tubes before milking. Redax strongly suggests Vaseline oil or cream to help the tube stripping or any other non-aggressive lubricant.

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Redax S.p.A.